


Intelligent ramp

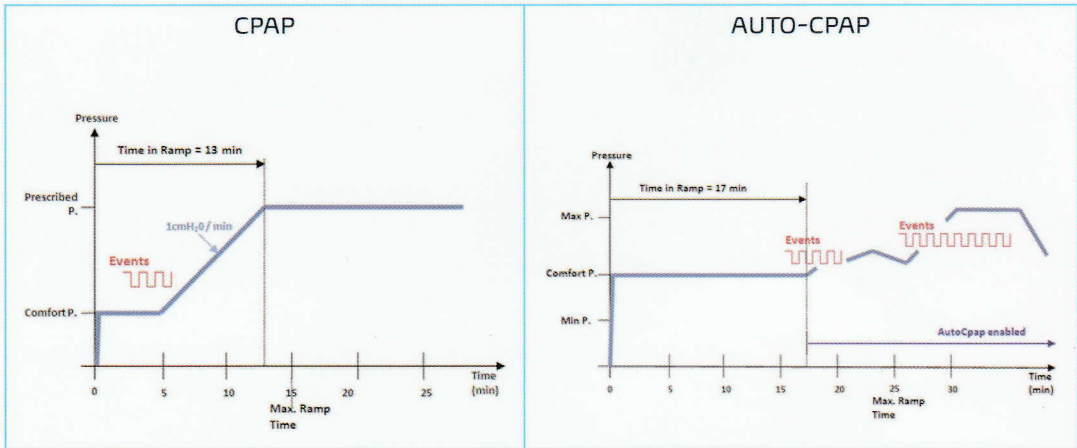

DreamStar Info/Auto settings

Button and browsing

The ◀ and ▶ buttons are used to:

- browse in the different menus,
- increase or decrease the value of a selected parameter.

The ▼/▲ button is used to:

- browse through a page from top to bottom,
- and select the settings.

The ⌚ button is used to:


- return to the top of the screen page by pushing it once,
- exit the menu by pushing again.



Access to the settings and maintenance menus: push the ◀ and ▶ buttons simultaneously for 2 seconds.

Setting	Auto-CPAP mode	Minimum value	Maximum value	Default value	Step
Comfort pressure		4 cmH ₂ O	Max. P. set	4 cmH ₂ O	0,5 cmH ₂ O
Maximum pressure on apnea		Min. P.	Max. P.	10 cmH ₂ O	0,5 cmH ₂ O
Maximum pressure		4 cmH ₂ O or Min. P.	20 cmH ₂ O	20 cmH ₂ O	0,5 cmH ₂ O
Minimum pressure		4 cmH ₂ O	20 cmH ₂ O or Max. P.	4 cmH ₂ O	0,5 cmH ₂ O
Ramp time (if ramp time is selected)		0 min	Max. ramp time	15 min	5 min
Maximum ramp time		0 min	45 min	45 min	5 min
Heat level (humidif.)		0	10	1	1

The comfort calibration CC+

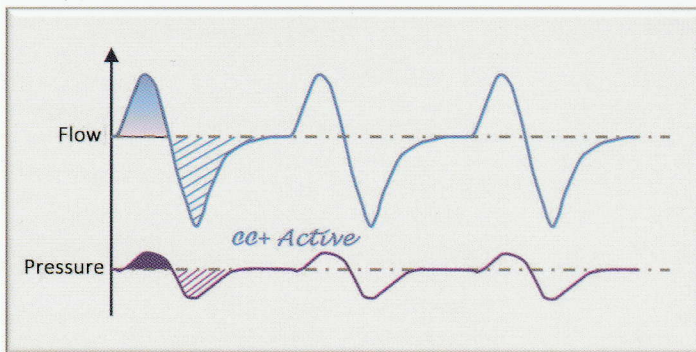

The purpose of the CC+ mode is to optimize patient comfort during CPAP therapy without compromising the stability of the upper airways at the end of exhalation and the beginning of inhalation.

CC+ mode **increases the pressure during inhalation and reduces it during exhalation**. These variations naturally adapt to the patient's respiratory cycles.

CC+ mode can be activated either during the ramp only, or during the entire therapy.