

Hidden button  
access to settings

Information  
access button  
(provides access  
to information)

Ramp button  
(lowers the value  
of the parameters  
display)



To access the device setting push the hidden key, then simultaneously the information access button for one second.

The symbol indicates that the parameter displayed can be modified.

To return to the previous data push the hidden button.

ON/OFF button  
(raises the value of the  
parameters displayed )

**Remark:** Exit from the parameter setting menu takes place automatically when you do not push any key for 30 seconds.

## EcoStar Info/Auto settings

APAP

**Mode:**

CPAP.  
APAP: Auto - CPAP.

10.0 cmH<sub>2</sub>O

**Pressure:**

In CPAP mode.

20.0 cmH<sub>2</sub>O

**Maximum pressure:**

In APAP mode.

4.0 cmH<sub>2</sub>O

**Minimum pressure:**

In APAP mode.

tr

**Ramp:**

Time ramp (tr).  
Indexed ramp (ir).

15 min

Ramp time in CPAP mode.  
Latency time in APAP mode.

6.0 cmH<sub>2</sub>O

Comfort pressure.

AO: 1

**Auto ON:**

The feature can be enabled (1)  
or disabled (0).

CC: 1

**Comfort calibration:**

The feature can be enabled (1)  
or disabled (0).

CP:HH 22

**Pneumatic calibration:**

Configuration 22 (tubing ø 22mm, standard)  
15 (tubing ø 15mm), HH (humidifier), SP  
(specific present) or - (specific absent).

FL: 1

**Command on Inspiratory Flow Limitation:**

In APAP mode only. The command can be  
enabled (1) or disabled (0).

Sd: F

**Pressure decrease speed:**

The speed - slow (S) or fast (F).

A 12.0 cmH<sub>2</sub>O

**Maximum pressure on apnea command:**

In APAP mode only.

1124h 39

**Blower hours:**

Device operating time in total.

738h 16

**Therapy hours:**

Time during which the patient actually  
breathed with the mask.

2000

**Pressure sensor offset:**

Integrated in the device.

RS: 0

**Selection of RS232 connection for an external module:**

The connection can be enabled (1) or disabled (0).

Id \*\*

**Software version:**

The version can be viewed but no modified.

## Specific calibration

- Put the device in standby
- Set up the desired pneumatic configuration (humidifier, patient circuit, etc...) with the patient interface.
- **Keep the patient circuit straight and the air outlet unobstructed.**
- Bring up the screen page of the setting menu related to «Pneumatic Calibration».
- Select the specific patient circuit CP:SP or CP:--:

Specific calibration present

CP:SP

Specific calibration absent

CP:--

- Push the hidden key and hold it down until launch of calibration taking care not to touch the pneumatic circuit.

## Setting the clock

- Put the device in standby.
- Push for 2 seconds, then keeping it hold down, push .








This screen appears:

8:29h

- Change the time displayed by using the or .
- Push to change minutes.
- Change minutes by using the or .
- Push to exit the menu.



## EcoStar Info/Auto Symbols

Symbol	Description	Symbol	Description
	Standby mode		Operating mode
	Ramp feature	<b>P</b>	Pressure symbol
	Time symbol		Compliance symbol
	Adjustable parameter	<b>Id *.*</b>	Software version
<b>In</b>	Information message	<b>Er</b>	Error message
	Pressure sensor offset		

Sefam

BR0123-03 / 05-2017

Factory: SEFAM - 10 allée Pelester Dossy - 54600 Villers-les-Nancy - FRANCE  
 Manufacturer: SEFAM - 144 avenue Charles de Gaulle - 92200 Neuilly sur Seine - FRANCE

technicalservice@sefam-medical.com

support@sefam-medical.com

customerservice@sefam-medical.com

Fax: +33 3 83 44 99 87

Tel: +33 3 83 44 77 40

Customer Service

After Sales Service

Technical Service



www.sefam-medical.com

TREATMENT



## EcoStar Info/Auto

Settings guide



Sefam